**CycleBeads**<sup>®</sup>  
Natural Effective Family Planning

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**CycleBeads**<sup>®</sup>  
Mpango Au Kuzuia Mimba Kawaida

U.S. patent No. 6,747,917 B2. Foreign patents pending.  
CycleBeads are distributed under license.

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**CycleBeads are a natural, effective approach to plan your family. Using CycleBeads can:**

- Help you understand your fertility.
- Help you know if your cycle length is appropriate to use the method to prevent pregnancy.
- Help you and your partner become more involved in family planning.

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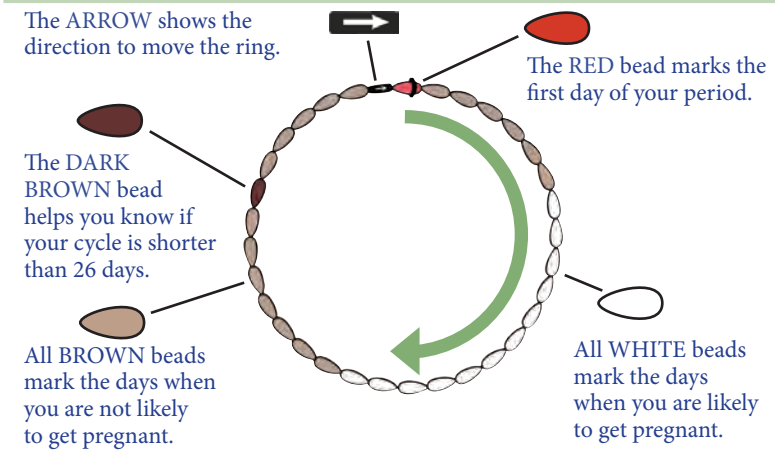
**What are CycleBeads?**

CycleBeads help you use the Standard Days Method<sup>®</sup> of family planning. When used correctly, the method is 95% effective in preventing pregnancy.

CycleBeads are a string of 32 beads, a rubber ring and a cylinder with an arrow.

**How do you use CycleBeads?**

The day you get your period move the ring to the RED bead.

Also, mark that day on your calendar.

Move the ring one bead each day. Move it even on the days when you have your period.

Abstain or use a condom when the ring is on any WHITE bead if you do not want to become pregnant.

You can have sex when the ring is on any BROWN bead. You are not likely to get pregnant on those days.

Move the ring to the RED bead again when your next period starts. Skip over any beads that are left.

**Who can use CycleBeads?**

- » Women who have most cycles between 26 and 32 days long.
- » Couples who can abstain or use condoms during the 12 white bead days, when the woman is likely to get pregnant.

A woman may need to wait a while before using CycleBeads if:

- she recently had a baby or is breastfeeding.
- she recently used the pill, the injection, or emergency contraception.

These situations can affect your cycle length. Talk to your healthcare provider.

**When to contact your healthcare provider**

- If you had unprotected sex on a WHITE bead day.
- If you think you might be pregnant because you have not gotten your period.
- If you get your period before you reach the DARK BROWN bead, this means that your cycle is shorter than 26 days.
- If your period does not start by the DAY AFTER you reach the last brown bead, this means your cycle is longer than 32 days.
- If more than once a year your cycle is shorter than 26 days or longer than 32 days, CycleBeads will not work for you. Ask your provider about other methods.

**If you are not sure if CycleBeads are right for you, contact your healthcare provider.**

CycleBeads do not protect against HIV/AIDS or other sexually transmitted infections.

CycleBeads are not a toy. Keep them in a safe place, away from children.

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